



FareTec Inc.
RESPONDER PRODUCTS INC.

EMERGENCY CARE | MILITARY | CIVILIAN



ABOUT US:

FARETEC INC.

Domestic Sales USA

FareTec, Inc. is a medical device manufacturer founded in 1991. Our facility is located in Painesville OH, 20 miles East of Cleveland. We specialize in devices for pre-hospital care both in the civilian and military markets.

For many years the company's primary focus has been on splints and various immobilization devices. The QD-4 hare style leg traction splint, has been an industry standard along with our ladder splints and utility splints. FareTec, Inc. has evolved in the 21st century by introducing the CT-EMS/CT-6 leg traction splint, LifeBox50 and the FS-3 Utility splint. These items represent the latest in leg traction, immobilization and insulation technology.

In 2009 FareTec Inc. launched their climate controlled containers line; the LifeBox50s. The LB50 has been designed to meet rising demands for administering cold therapy treatment immediately following cardiac arrest. The FastSet-3 was also developed in 2009 to provide a more rigid immobilization alternative to the market. This same year, the FastSet-3 won the EMS "Top Product" Innovation award.

In 2011 FareTec Inc. added a Wheeled Litter Carrier, the EL3000, to their product line. It was named JPOC (Joint Product of Choice) by the US military and is quickly becoming a staple for patient transport in both the US Military and Civilian Markets. In 2013, the EL2000 won the EMS "Top Product" Innovation Award.

RESPONDER PRODUCTS INC.

International Sales

Established in 2005, Responder Products was born out of a need for small to medium sized companies to utilize joint resources to facilitate growth into target export markets. As a result, Responder Products has developed strong relationships with distributors around the globe to enhance several companies' international marketing potential. Most notably, H&H products and their Bolin Chest Seal; used in our Individual First Aid Kit, is becoming world renowned. Responder Products maintains their focus on: EMS, Military, and Search & Rescue in various target markets.



CONTENTS:

PAGE 3: CT-EMS Carbon Traction Leg Splint (Civilian)
PAGE 4: CT-6 Tactical/Military Carbon Traction Leg Splint
PAGE 5: EL2000 Civilian Patient Transport Device
PAGE 6: EL3000 Military Patient Transport Device (WLC)
PAGE 7: LifeBox50 Civilian; Temperature Controlled Container Series
PAGE 8: LifeBox50 Military; Temperature Controlled Container Series
PAGE 9: QD-4 Adult & Pediatric HARE Traction Leg Splint
PAGE 10: FareTec Redi Splint & Wire Ladder Splint
PAGE 11: FastSet-3 Fiberglass Moldable Utility Splint
PAGE 13: H&H Bolin Chest Seal
PAGE 14: H&H Chest Wound Kit & Combat Eye Shield
PAGE 15: H&H Compression Bandages
PAGE 16: H&H Combat Cravat & TK-4L Tourniquet
PAGE 17: H&H Specialty Kits
PAGE 18: Combat Gauze
PAGE 19: Individual First Aid Kit (IFAK)
PAGE 21: Responder Backpack Series
PAGE 22: Curaplex Adjustable Extrication Collars & Finger Splint Kits

FareTec Inc. & Responder Products are committed to customer service and quality products. If you have any questions please call or email us at any time. Our products can be purchased from your preferred distributor.

Follow us on Twitter: @FirstResponder

www.FareTec.com

www.ResponderProducts.com

Product Videos and Manuals Available

CAGE CODE: 0R123

DUNS: 626802540

P: 800.322.3273 or 440.350.9510

F: 440.350.9520





TRACTION

CT-EMS

CARBON LEG TRACTION SPLINT TECHNOLOGY

The CT-EMS is a Leg Traction Splint designed for Pre Hospital Care and Patient transport. It is primarily used on patients with mid-line femur fractures. The CT-EMS helps relieve patient pain and prevents further damage to surrounding muscle and tissue while reducing the risk of death caused from severed arteries. The CT-EMS is made of carbon tubing which snaps together via an internal bungee. Traction is achieved with a 4 to 1 pulley system using a small line to apply tension. This method results in a very precise and delicate form of traction that is also powerful enough to be effective on extra large adults. The CT-EMS is extremely compact, light and strong.

FEATURES:

- Carbon Fiber Tubing: Corrosion resistant, temperature stable, extremely high strength to weight ratio, x- ray translucent.
- 4:1 Pulley Traction System: precise traction for adults and pediatrics. The V- Jam cleat makes it easy to readjust traction, and includes safety clip to prevent accidental releases.

PART No.

CT-EMS 1126626

Ankle Hitch 1131020

Leg Straps 1131045

Carry Case 1131060



SPECIFICATIONS:

Size/weight:

- -3 x 4 x 12" (7.6 x 10.2 .30.5cm)
- -1 lb (500 grams)

Colors:

- Orange Straps, Black Carbon
- Dimensions: 3 x 31 4 x 36

Materials:

- Carbon fiber tubing
- Nylon webbing
- -Aluminum/Plastic buckles





TRACTION

CT-6

TACTICAL LEG TRACTION SPLINT TECHNOLOGY

The CT-6 has been designed for tactical and military use. The main difference between this and the CT-EMS is the color layout of the materials. The CT-6 and all of its straps are black and on the military version, the Ischial strap utilizes a Cobra buckle.

Over 25,000 units currently in use around the world in various militaries. This product is extremely well tested. Each CT-6 contains everything that is needed, including synch bag, instruction manual, ankle hitch and leg straps; all of which are easily replaceable with their own part numbers.

FEATURES:

- The CT-6 comes out of the bag ready to go; all of the necessary parts are attached to the splint. There is no way to lose pieces, resulting in very fast application times.
- New Ischial Strap Design.
- Upgraded Cobra "Side Release" buckle and new elastic padding.
- Molle compatible bag.



PART No. and NSN No.

CT-6 Tactical

Part 1126624

Military NSN No.

CT-6 NSN 6515 01 521 5730

Ankle Hitch 6515 01 522 1717

Leg Straps 6515 01 523 5599



- Instruction Manual: Each splint comes with its own "quick reference" two sided instruction manual that is water proof and tear resistant.
- Made in USA; guaranteed quality product. CE approved.





TRANSPORT

EL3000 MILITARY

NATO WHEELED LITTER CARRIER

The Litter Carrier is a light, stable device, equipped with two large non-pneumatic wheels designed for transporting injured persons on rugged terrain. These flat-free tires keep the unit mobile at all times, without the need for an air pump. When the supporting legs are in the transport position the patient is easily moved by one or two persons. When the supporting legs are in the static position, the litter carrier is a steady and stable support for the patient. When not in use, the Litter Carrier is easily folded to a compact size for storage. The EL3000 Military Litter Carrier has proved very useful for transporting and moving injured patients (or materials) inside and outside field hospitals. It also provides access to AMPs, helicopter pads and airlift areas in typical large-scale emergency or natural disaster scenarios.

FEATURES:

- Transport patients or supplies quickly with just one bearer, over a variety of terrain. This will save personnel time and energy in the field.
- Metal Hand Wheel system: Allows for quick adjustments that will secure any sized litter or backboard. Intuitive design allows for one handed application.

PART No. and NSN No.

Part 221030

NSN 6530 01 591 9636

The Joint Product of Choice (JPOC) for Wheeled Litter Carrier as deemed by the Defense Medical Material Program Office (DMMPO)

GSA Contract: GS-07F-0018W

ECAT & DAPA Compliant



SPECIFICATIONS:

- Weight: 21Kg (46.29 lbs.)
- -Dimensions: Open L x W x H: (158 x 62 x 86cm)
- -Dimensions: Closed L x W x H (52 x 62 x 84cm)



TRANSPORT

EL2000 CIVILIAN

PATIENT TRANSPORT DEVICE

Currently in service by all branches of the US Military and foreign military units around the world; this product is extremely well refined and tested. Its versatility lies in its ability to fit all sized litters and backboards. The EL2000 is also used by the National Forest Service, the Federal Prisons Bureau, airports, fire stations and other civilian medical facilities across the country.

The following accessories are available for military and civilian models:

- IV Pole, carbon fiber
- Exam Light, LED
- Mayo Tray, height adjustable
- Storage Bag
- Arm Board, removable padding
- Dual Wheels, two per side.

FEATURES:

- Heavy duty wheels include solid inserts to prevent flat tires. These are equipped with quick release pins that allow the carrier to be assembled and disassembled in under 30 seconds.
- Temporary Hospital Bed: when the legs are in static position, the unit functions as a stable working platform for medics.

AREAS OF USE:

- Disaster Tents & Mobile Hospitals
- Sport Venues and Stadiums
- Airports
- Indoor & Outdoor Concert Venues
- Street Festivals
- Crowd Scenarios
- Forests (Hunting)
- Parks (Emergency Evacs)
- Beaches & many more.



SPECIFICATIONS:

- Powder coated steel top coat & anti-corrosion base layer paint process.
- Weight Limit: Current max. test level, 500 lbs.
- Stainless steel fittings.
- Assembles/Disassembles in under 30 seconds.



COOLING

LIFEBOX50

CLIMATE CONTROLLED CONTAINER SERIES

The next generation of EMS transport containers is here. The LifeBox50 is extremely rugged, requires no power, and can maintain temperature for 30-50 hours. The key lies within the LB50 carbon vacuum insulated panels that produce insulation values of R50; 2x more powerful than anything else on the market. This allows for more payload space and greater efficiency in both hot and cold environments.

The LifeBox50 is designed for the First Responder to transport anything they need that is temperature sensitive; without having to worry about a heavy, tethered powered unit. The LB50 is ideal for transporting Cold Therapy items essential for inducing hypothermic therapy. Additionally, they work great transporting blood units, saline, body parts and medication packs.

FEATURES:

- Rugged Pelican Storm Case shell.
- R50 Vacuum Insulate Panels.
- Digital temperature display, battery and solar powered.
- Equipped with digital thermometer display, for payload temperature monitoring. Solar/Battery powered.

PART No.

LB50 LG: 6001024

LB50 MD: 6001004

LB50 SM: 6001014

Briefcase: 6001023

Chill pack: 6002040



- Temperature data recording options available.
- Internal USB DATA Loggers and other programs can be implemented to fit customer requirements.
- Patent Serial No. 29/316,628, other patents pending.



COOLING

UNIT INCLUDES:

- Solid Phase Change gel packs
- Digital Thermometer
- Instruction Manual
- Carry Strap
- Available in several sizes and colors. Custom units available.

NSN No.

SMALL LB50

NSN#: 8145 01 596 4629

STANDARD L50

NSN#:6545 01 596 4015

LIFEBOX50

CLIMATE CONTROLLED CONTAINER SERIES

The LifeBox50 will maintain what's essential to you at its appropriate temperature. The LB50 eliminates restraints caused by temperature changes over time. The insulation technology, combined with the Pelican Case design, allows each unit to maintain its core temperature for exceptionally long durations. The LB50 has been tested in the harshest environments by both emergency services and the military.

The LifeBox50 uses passive cooling sources which require zero energy from a power supply. This eliminates the possibility of a motor failure or breakage of mechanized parts used for cooling. Each LB50 is mounted with a digital thermometer to monitor its core temperature. These unique features make our LifeBox50 the number one choice for any medic.



LifeBox50 Specifications	Standard	Large	Small
Payload Capacity (inches cubed)	684	950	385
Payload Dimensions (inches)	14.4 x 9.5 x 5	10.4 x 15.4 x 6	11 x 3.5 x 10
Exterior Dimensions (inches)	19.2 x 15.2 x 9	21.2 x 16 x 10.6	16.9 x 10.9 x 13
Tare weight (lbs)	12.1	19.04	8.52
Outer Container	Rugged Pelican/Storm Case. Colors: Black or Olive-Drab	Rugged Pelican Case w/wheels and Tow Handle. Colors: Black or Olive- Drab	Rugged Pelican Case w/shoulder strap. Colors: Black, Desert Tan, Olive-Drab, Orange, Yellow
Insulator	Carbon Aerogel Vacuum Panels	Carbon Aerogel Vacuum Panels	Carbon Aerogel Vacuum Panels
Digital Thermometer	Yes/battery & solar powered	Yes/battery & solar powered	Yes/battery & solar powered
Recommended Use	Blood: 8-10 units, Cold Therapy: 4 units Saline, Medications	Blood: 12-14 units, Cold Therapy: 6 units Saline, Medications, Tissue/Organs	Blood 4-5 units, Cold Therapy: 2 units Saline, Medications



TRACTION

QD-4 HARE ADULT & PEDIATRIC TRACTION SPLINT

Used by emergency services across the country, the QD line of splint kits has several unique and proprietary features. The "Hex Lock" system allows the splint to be quickly adjusted to any length. Once it has been sized appropriately, the intuitive locking mechanism is engaged by turning the Hex Lock to either side. It's fast, easy and reliable. This system helps eliminate problems with dirt or corrosion which cause other splints to malfunction. It has become an industry standard for patients requiring Hare style traction splinting.


The QD-4 splint will adjust to accommodate pediatrics and adults. All units come as a complete kit with straps, ankle hitch and carrying case included. In addition, all QD's are available with either our anodized aluminum ratchet or our injection molded ratchet.

FEATURES:

- Compact, Strong, Light.
- Aluminum or Molded Ratchet.
- Integral Strap System.
- Field tested.
- QD-4 fits Pedi to Adult.
- Intuitive Adjusting System.
- Minimal patient movement.
- QD-3 Pediatric traction splint for states that require separate unit.

SPECIFICATIONS:

- QD-4 Length Extended 54" (137cm)
Length Closed 32" (81cm)
- QD-3 Length Extended 45" (114cm)
Length Closed 29" (73cm)
- Weight: 10.4 lbs (4.7 kilos)

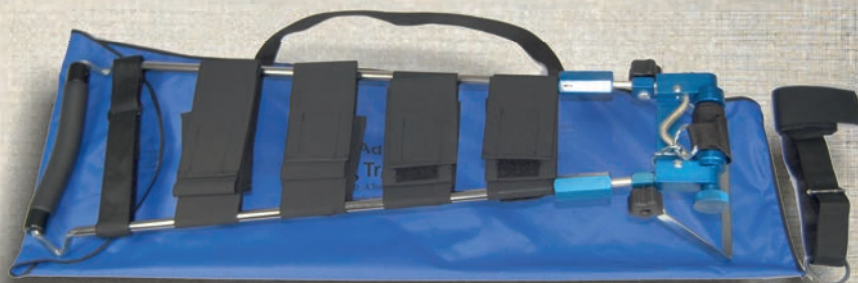

Part Number	Description	Ratchet
1126524	QD-4 Adult Splint	Aluminum
1126522	QD-3 Pediatric Splint	Aluminum
1126514	QD-4 Adult Splint	Molded
1126512	QD-3 Pediatric Splint	Molded
1126726	QD 3&4 Combination	Aluminum
1126716	QD 3&4 Combination	Molded





SPLINTING

FEATURES:

REDI SPLINT

- Strong and supportive for any fractured limb.
- Extremely pliable.
- Can be cut with scissors.
- Field tested.
- Available in various colors/sizes.

WIRE LADDER

- Thick coating for safety & comfort.
- Pliable for easy right angles.
- Gauze or a triangular bandage.
- Can be used in multiples for added rigidity.

PART No.

WIRE LADDER 1211300

REDI SPLINT 1211306

REDI SPLINT

FARETEC UTILITY SPLINT

The Redi Universal Splint is a moldable utility splint that can be used on a wide range of injuries. It provides strong and supportive immobilization for fractured limbs. The splint uses "dead soft" aluminum that allows it to be molded as necessary.

Unlike many utility splints on the market, the Redi-Splint uses closed cell, cross linked Volara Foam. This foam will not irritate the skin and is easy to clean


Used around the world by: EMS Personnel, Military Medics, Athletic Trainers, Ski Patrols, Wilderness Rescue, and many more.



WIRE LADDER SPLINTS

This staple of the EMS Industry is alive and doing well. Still manufactured in large numbers, the FareTec Wire Ladder Splint is coated with a thick epoxy coating to cover all its edges. These are pliable and can be formed to any shape.

The FareTec Ladder Splint is one of the most economical forms of utility splint on the market today.

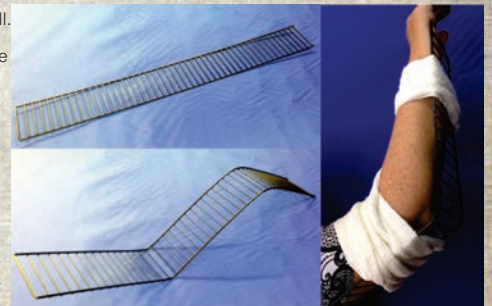

SPECIFICATIONS:

- | | |
|-------------------------------|------------------------------|
| • Wire Ladder Size: | Redi Splint Size: |
| • 31" x 3.75" | 4" x 36" (7.6 x 78.7 cm) |
| • (78.7 x 9.5 cm) | Sold/Packaged Flat or Folded |
| • Sold/Packaged by the dozen. | |



SPLINTING

FASTSET-3

FIBERGLASS MOLDABLE UTILITY SPLINT

The FastSet-3 provides patients with the ideal orthopedic splinting experience. It combines the rigid, supportive properties of a cast with the "ease of use" of a utility splint. The FS-3 can immobilize a patient moments after the first responder arrives. The FS-3 is compact and light weight, making storage convenient for all types of responders. The splint can be rolled or folded until its time of use. Once the fiberglass splint is opened and activated, it provides the means to immobilize almost any fracture. The splint is simply wrapped on the injured limb and then activated with 1-2 ounces of liquid. The splint sets in about 3 minutes. The future of utility splinting is now. The FS-3 offers a "Complete Splinting Solution".

FEATURES:

- Water activated polyester.
- X-Ray translucent.
- Sets in just under 3 minutes.
- No Gloves necessary.
- Safe for direct contact with skin
- Secure, Support, Immobilize.

PART No.

FS-3 3"x12" Case (5)

Part 1211500

FS-3 4"x15" Case(5)

Part 1211502

FS-3 4"x30" Case (5)

Part 1211504

FS-3 5"x30" Case (5)

Part 1211506



SPECIFICATIONS:

Sizes:	Weight
3 x 12in. (7.6 x 30.5cm)	.35lbs / 159grams
4 x 15in. (10 x 38cm)	.43lbs / 195grams
4 x 30in. (10 x 76cm)	.82lbs / 371grams
5 x 30in. (12.7 x 76cm)	.98lbs / 445grams



SPLINTING

FASTSET-3

FIBERGLASS MOLDABLE UTILITY SPLINT

The FS-3 is conveniently packaged in a sealed, easy to open bag. This item is also available in a complete kit. The kit includes one FS-3 Splint, a Responder Gauze bandage and a Saljet; for easy wrap up and activation. The Splint can be easily applied by remembering the following three step process: REMOVE the splint from the packaging. WRAP the splint onto the patient. WET the splint to speed the hardening process.

Military Kit includes extra durable, tactical packaging with instructions and easy open pull holes.

FEATURES:

- Sealed, durable compact bag.
- Quick & easy open tear holes.
- Available boxed or individual kits.

Each FS-3 size is available in a separate kit featuring:

- (1) Responder Gauze
- (1) FastSet-3
- (1) 30ml Saljet



SPECIFICATIONS:

- Packaging: Airtight/watertight, heavy duty wrapping.
- Quick open tear holes.
- Materials: Moisture wicking synthetic fabric, water activated fiberglass.



BOLIN CHEST SEAL (BCS)

STERILE OCCLUSIVE CHEST WOUND DRESSING

The BCS is specifically engineered to meet the critical needs of sucking chest wounds. It treats open pneumothorax and preventing tension pneumothorax that result from gunshots, stab wounds, and other penetrating chest trauma. The BCS allows air and blood to escape while preventing the re-entry of either, thereby eliminating any unwanted gas or liquid exchange at the trauma site.

The large (6" diameter), rugged polyurethane disc structure of the BCS can cover practically any size chest trauma site. The temper of the disc prevents any disc wrinkling from occurring during application. The wound side of the disc is covered with a strong adhesive that can seal over hair and blood. The BCS is flexible enough to be removed and reapplied to the trauma site if required.

BOLIN CHEST SEAL

FEATURES:

- NEW Inline Triple-Valve design.
- NEW Latest high adhesion Hydrogel: provides a thick layer of gel-based adhesive.
- Large pad for wiping down wound site.
- Can be rolled for easy kit storage.
- XL Size available.

PART No. and NSN No.

Item Number: BCS-01

NSN: 6510-01-549-0939

BOLIN XL

6510-01-599-2727



SPECIFICATIONS:

- Package Dimensions: 7.5" x 7.5" x .25" (19x19x0.6cm)
- Package Weight: 3 oz.
- US Patent No. 7,834,231
- 3 year shelf life





COMBAT EYE SHIELD

PROTECTING PENETRATING EYE TRAUMA

H&H's Combat Eye Shield has been designed according to the specifications of several active duty surgeons of OIF/OEF to cover penetrating eye injuries. The Combat Eye Shield is designed to meet the CoTCCC guidelines for the treatment of penetrating eye trauma while providing a small, sterile package to fit in any trauma kit.

COMBAT EYE SHIELD

FEATURES:

- Sterile and vacuum sealed.
- 3 year shelf life.
- NSN: 6515-01-590-2668

Product Dimensions:

- 4 x 2 x 1/4 (10x5x0.6cm)
- - Product Weight: .5 oz
- - Part# 2221033
- - Design Patent No. D674,903



The Combat Eye Shield is composed of a six inch Hydrogel disc that surrounds a fox eye shield, giving a large surface area to attach to the casualty that will not harm adjacent trauma or burns.

The Combat Eye Shield also comes with one (8 x 7.5 ABD) pad to clean the surface of the wound area before applying the shield.

WOUND SEAL KIT

WOUND SEAL KIT

FAST APPLICATION FOR SEALING WOUNDS

The H&H Wound Seal Kit has been designed for sealing wounds quickly to stop external bleeding. The Wound Seal Kit is a sterile, TCCC-compliant occlusive dressing that comes vacuum sealed with a 6 x 8" plastic sealing rectangle and hydrogel adhesive to stick to the wound and the surrounding skin through blood and debris. Along with the wound seal, the kit comes with a sterile (8 x 7.5 ABD) pad. The sealing rectangle has the same polyurethane carrier and adhesive as the Bolin Chest Seal.

SPECIFICATIONS:

- New redesigned seal comes with rounded edges to reduce friction with clothing.
- New long pull tab to allow for venting, easy removal and repositioning of the seal over the injury.
- 3 Year Shelf Life





H-BANDAGE

UNIVERSAL SEVERE TRAUMA DRESSINGS

The H-Bandage gives medics and field personnel both an emergency dressing and a hemostat for arterial bleeding and hemorrhaging in limbs. The H-Bandage's unique design allows for one-handed use under any condition. Designed for First Responders, Military, and Law Enforcement, the H-Bandage provides a fast, effective tool for serious traumas. The H-shaped cinch allows for direct pressure to be applied directly on top of a wound, increasing compression and reducing blood loss. The H-Bandage can be used easily and effectively as a field compression dressing and as a sling with improved stability, compression, and ease of use. Clinically tested to stop severe bleeding; a versatile combat dressing

H-BANDAGE

FEATURES:

- Folds Flat: 7.5 x 4.5 x .75in.
- 8 x10" (20x25cm) ABD Pad
- 5ft (1.5m) Elastic Webbing
- H Shaped Cinch
- Package Weight: 3 oz.
- Shelf Life: 3 years



RESPONDER BANDAGE

CE MARKED

This item is designed to have the same features and benefits of the H-Bandage. Responder Gauze has been registered and labeled accordingly with CE regulations for the sale into Europe and nations around the world.

SPECIFICATIONS:

H-Bandage Hemostat Dressing

- NSN 6510-01-540-6484
- Part# 2221003

Responder Bandage, CE Marked

- Part# 2221098
- Big Cinch Abdominal Bandage
- Size: 12"x16" (30.5x40.6cm)
- Part# 2221005





COMBAT CRAVAT

DRY STERILE BURN DRESSING

A burn dressing and cravat, this multi-purpose tool fits easily into any aid kit. By making a dry sterile burn dressing into the shape of a large cravat, first responders have a product that addresses several issues. This First Aid providers have a product that works as a burn dressing, tourniquet, sling and has numerous other capabilities.



COMBAT CRAVAT

FEATURES:

- Spunbound polypropylene.
- 2" Reinforcement seams, lock stitched.
- Added .5" twill trim surrounds cravat.
- Meets TCCC guidelines for burns.
- Sterile, non-adherent.
- Vacuum packaged in 3mil nylon poly.
- Includes two #3 closed safety pins and a rubber band.

SPECIFICATIONS:

- Packaged: 4.25x 4.2 x 0.5 (10.8 x 10.8 x 1.3cm)
- Opened: 45 x 63in (114 x 160cm)
- Product Weight: 2 oz
- NSN: 6501-01-587-6579
- Part# 2221045



TK-4L

TOURNI-KWIK TK-4L

ONE HANDED TOURNIQUET

The TK-4L is a simple to use device that makes stopping arterial bleeding easy and direct for injuries ranging from combat trauma to accidental injury. It features steel hooks fastened steadfast to both ends of an elastic band, and packaged in a small, vacuum-sealed pouch. TK-4L has been deployed by the US Marine Corps, US Special Forces, and numerous law enforcement and emergency services around the world.

FEATURES:

- Woven elastic textile band (3x40in).
- Provides strength and flexibility.
- Ability to apply maximum capable force on lower extremities.



SPECIFICATIONS:

- Package: 2.5 x 2.5 x .6
(6.4 x 6.4 x 1.5cm)
- Length: 40" (101cm)
- Package Weight: 4 oz.
- NSN: 6515-01-578-6354
- Part# 2221037



H&H EMERGENCY CONVENIENCE KITS

These kits contain the tools necessary to perform a variety of common procedures that occur in emergency situations. These kits provide the essential items in a compact vacuum-sealed package. Each item is light weight, sterile, single use and features an easy open tab. Many of these have been designed using specifications from OIF/OEF active duty surgeons.

CONVENIENCE KITS

PART No. and NSN No.

Chest Tube Insertion Kit
NSN 6515-01-579-3275
Part 2221070

Cricothyrotomy Kit
NSN 6545-01-599-2818
Part 2221018

Lateral Canthotomy Kit
NSN 6515-01-600-4521
Part 2221019

Situational Instrument Tray Kit
NSN 6515-01-590-1339
Part 2221073

Burn and Anti-Infection Kit
Part 2221021


Penetrating Chest Injury Kit
NSN 6515-01-590-7855
Part 2221026

The H&H Emergency Chest Tube
Insertion Kit




The H&H Emergency
Cricothyrotomy Kit

The H&H Situational Instrument Tray (SIT) Kit

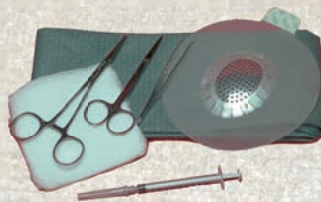

The H&H Lateral Canthotomy Kit

The H&H Penetrating Chest Injury Kit

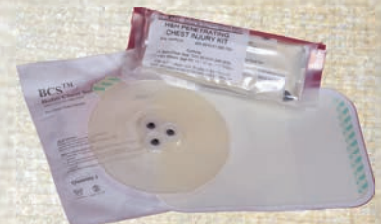



PRIMED COMBAT GAUZE

GAUZE



This item addresses moderate to severe wounds involving heavy blood loss. Made of high quality cotton for maximum absorption ability. H&H PriMed Gauze is vacuum packaged small for easy storage yet expands to cover large wound areas. It is the perfect companion to our compression bandages to stop severe blood loss and save lives.

FEATURES:

- High quality cotton.
- Vacuum packed.
- Multiple uses.
- Field tested.



RESPONDER GAUZE

CE MARKED COMPRESSED GAUZE

This item is designed to have the same features and benefits of the PriMed gauze. Responder Gauze has been registered and labeled accordingly with CE regulations for the sale into Europe and nations around the world. Available in new silver foil package or standard poly bag.



SPECIFICATIONS:

- NSN# 6510-01-503-2117
- Part# 2221038
- Package Dimensions: 2.5x2x0.5" (6.4x5x1.3cm)
- Weight: 1oz.
- Shelf Life: 3 Years



RESPONDER IFAK

INDIVIDUAL FIRST AID KIT

An Individual First Aid Kit; packaged small enough to be carried anywhere, rugged enough for even the most extreme environments, sterile and ready for immediate use with a minimum amount of training. Responder IFAK kits contain only premiere quality products, all of which are rigorously tested in the field by groups around the globe.

Each unit contains the following: Bolin Chest Seal, Compression Bandage, Combat Medic Tape & Gauze, Tourniquet, Gloves, Nasal Pharyngeal, IFAK Bag. The contents can be altered to fit customer needs; we build to spec.

FEATURES:

- Molle equipped bag, several styles.
- Elastic product loops.
- Custom product designs and layouts.
- Field tested equipment.
- Tight wrap, compact designs.
- Several material variations.

PART No. and NSN No.

FareTec IFAK

Part# 3331000

IFAK BAG

Part# 3331001

Includes pullout insert/cord



SPECIFICATIONS:

- Available Colors: Black, Coyote Brown, ACU Digital and others.
- Size: 6.5" x 5" x 3.5" (16.5 x 12.7 x 8.9cm)
- Weight: 1 lb 1 oz (0.5 kilo)
- Bag style varies to fit customer requirement





RESPONDER IFAK

INDIVIDUAL FIRST AID KIT

The below items are recommended by FareTec Inc and Responder Products for placement in the combat IFAK. These items were chosen based on ISR (Institute of Surgical Research) and the feedback received from those in the field. All items are subject to change at the customers request.



CAT Combat Application Tourniquet:
NSN 6515-01-521-7976
Part # 3333000



Responder Compression Bandage, CE marked:
Part # 2221098

Bolin Chest Seal (BCS):
NSN 6510-01-549-0939
Part # 2221025 CE Marked



Combat Tape:
NSN 6510-01-549 0927
Part #2221017

Responder Gauze, CE marked:
NSN 6510-01-503-2117
Part # 2221038





RESPONDER FIELD MEDPACK

BACKPACKS TO SUPPORT YOUR NEEDS

This bag is an excellent piece of gear for military, law enforcement and civilian medical personnel who need a convenient way to carry an extensive array of medical supplies and equipment into the field. The pack is designed with multiple interior compartments and panels, each loaded with pockets, straps and other attachment points. Open compartments are designed to lie flat, providing full access to the necessary supplies.

The Responder MedPack is made from heavy duty water-resistant material and incorporates a padded back that promotes air flow and contoured adjustable shoulder straps for comfort. It has a waist belt and compression straps as well as a grab handle on the top and side storage pockets.

FEATURES:

- Regular and Pro Models.
- Cordura1050 denier fabric.
- YKK high strength zippers and tracks.
- UTX-DURAFLEX buckles with low sound closures.
- High tensile strength nylon webbing.
- High tensile strength composite nylon thread.
- High tensile strength nylon velcro.

Colors: Black, Olive Drab, Desert Tan
Custom units available.



MODELS & SIZING:

Responder Pro 44L: 58x38x20cm Part# 7001440
Responder 44L: 58x38x20cm Part# 7001044
Responder 44 LT: 65x30x25cm Part# 7001004
Responder 34L: 48x35x20cm Part# 7001034
Responder 19L: 40x27x18cm Part# 7001019
Responder Pro 11L: 43x25x10cm Part# 7001110
Responder 11L: 43x25x10cm Part# 7001011



CURAPLEX COLLARS

ADJUSTABLE EXTRICATION COLLARS

The Curaplex Extrication Collar is an adjustable, single-use C-collar. It is available in an adult or pediatric (“mini”) size, accommodating a wide range of patients. Latches allow for accurate sizing and two safety lock buttons hold the collar in place once adjusted. The radiolucent collar is CT and MRI compatible. It is packaged flat for easy storage.

- Collar-Adult/sold 900/pallet
- Collar-Pediatric/sold 900/pallet
- Part# 4003010
- Part# 4003020

FEATURES:

- Accommodates a wide range of patients with two collar sizes.
- Automatic flip up chin and cannula hooks to store oxygen tube.
- Applies to patients in three easy steps – sizing, locking and placement.
- Folds flat for easy storage.



- Single use
- Adjustable
- Six settings

FINGER SPLINT KITS

This kit address one of the most common injuries a medic will encounter. The FareTec Finger Splint Kit immobilizes and protects dislocated joints and fractured phalanges. These are foam lined, aluminum splints that contour to the finger. Choose from several styles to best treat your patient.

FEATURES:

- Ships in a durable clear plastic container with hinged cover.
- Styles include: Cutter, Baseball, Finger Cot, 4-prong and Frog.



- Part # 1106701
- Several Sizes of each style



FareTec Inc.

RESPONDER PRODUCTS

1610 West Jackson St. #6
Painesville OH 44077
USA
440.350.9510
www.FareTec.com
www.ResponderProducts.com



H&H Medical Corporation
Critical Care Solutions