ENGLISH

SPANISH

GERMAN

ITALIAN

FRENCH

DUTCH

PORTUGUESE

**ENGLISH**  
BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

**INTENDED USER PROFILE:** The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

**INTENDED USE/INDICATIONS:** For immobilization of soft tissue injuries, fractures or dislocation of the distal interphalangeal joint.

**CONTRAINDICATIONS:** None

**WARNINGS AND PRECAUTIONS:**

- If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.
- If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.
- Never wear the support over an open wound.
- Do not use this device if it was damaged and/or packaging has been opened.

**APPLICATION INFORMATION:**  
1. Place splint on finger in desired position and secure with tape or straps.

**CLEANING INSTRUCTIONS:**

Hand wash liner in warm water using mild soap, rinse thoroughly. AIR DRY.

**NOTE:** If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

**WARRANTY:** DJO, LLC will repair or replace all or part of unit and its accessories for material and workmanship defect for a period of six months from the date of sale.

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